


TECHNICAL DATA

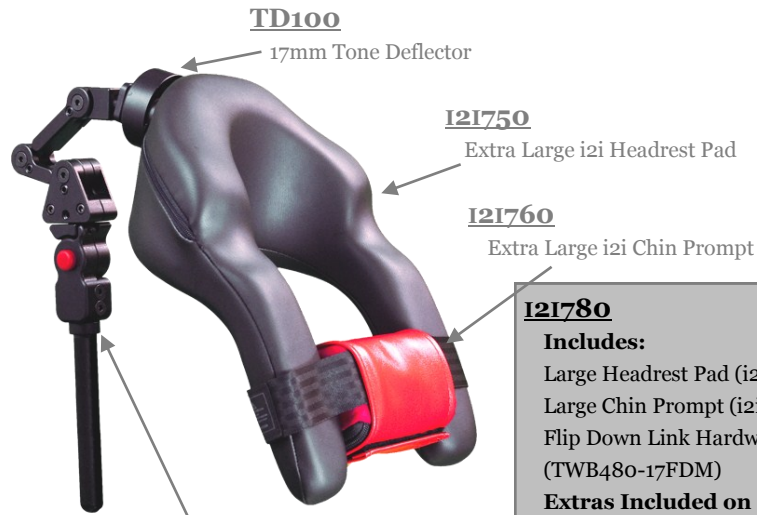

TWB480-17FDM

17mm Linking Hardware w/ Flip Down

- Includes:**
 Large Headrest Pad (i2i750)
 Large Chin Prompt (i2i760)
 Flip Down Link Hardware (TWB480-17FDM)
Extras Included on Left:
 Tone Deflector (TD100)
 Neoprene Glide Skin Black (c55)



DIMENSIONS

Product Number	Overall Width	Maximum Neck Circumference	Maximum Neck Width	Posterior Height	Arm Length
i2i 280 Niño X-Small	7"	7"	3"	3"	4½"
i2i 380 Niño Small	7½"	9"	3½"	3½"	4½"
i2i 480 Small Standard	8"	11"	4"	4"	6½"
i2i 481 Small Short Arm	8"	11"	4"	4"	4½"
i2i 580 Medium Standard	9"	14"	4½"	4¾"	7½"
i2i 581 Medium Short Arm	9"	14"	4½"	4¾"	5½"
i2i 680 Large Standard	9½"	16"	5"	5¼"	8"
i2i 680 Large Short Arm	9½"	16"	5"	5¼"	6"
i2i 780 X-Large Standard	10"	18"	5¾"	6"	9"
i2i 781 X-Large Standard	10"	18"	5¾"	6"	6¼"

PRODUCT DESCRIPTION

The i2i has been one of Stealth's signature products and one that is recognized around the world. The i2i head, neck and shoulder positioning system is offered in eleven different series. We have made significant improvements through the years and now have added a few more. It's important to remember that the Headrest is the finishing point of any seating system—not a starting point. If a client does not hold his/her head erect it does not necessarily mean he has poor head control. *It could be the result of a poorly fitting or improper seating system!*

TECHNICAL DATA

- 5/32 tool for socket-head and flat-head screws on mounting ring and mounting hardware.
- Includes: ball mounting ring and headrest pad.
- Headrest attaches to the TWB480-FDM.

SELLING POINTS

- The i2i is designed to help those with difficulty of movement and keeping the head upright.
- Keeping the head straight and level improves eye contact to promote socialization and assist in interpreting responses and communication.
- Keeping the user's head straight eases feeding, reduces saliva drool and greatly reduces the need to reposition the user's head, neck, and shoulders.

BUILD YOUR I2I SERIES HEADREST

STEP 1: NECK SIZE

The individual's neck circumference will determine the appropriate size i2i.

NECK SIZES

480	Under 11"
580	11"-14"
680	14"-16"
780	16"-18"

STEP 2: ARM LENGTH

To determine arm length, measure from the top of the clavicle to the bottom of the sternum.

ARM LENGTHS

i2i Standard

480	6.5"
580	7.5"
680	8"
780	9"

i2i Short Arm

481	4.5"
581	5.5"
681	6"
781	6.25"

STEP 3: EAR CUTOUTS (I2I-EC)

Individuals with lateral flexion will need ear cutouts and gel inserts to help reduce friction and pressure points.

STEP 4: FABRICS

Select a material for both the i2i and the Chin Prompt.

FOAM/GEL

Closed Cell Foam
Action Gel

COVERS

Dartex (standard)
Reverse Dartex
Vinyl
Sheep Skin
Neoprene

STEP 5: LINKS (TWB-LINK)

Standard i2i Hardware has a maximum reach of 6". Additional links increase the maximum reach.

STEP 6: SWING-AWAYS (SUS9 SERIES)


Facial lateral Swing-Aways can be added to the i2i for pad, switch, speaker sites and chin control devices.

HARDWARE OPTIONS



TWB480-17FDM

An easy transfer flip down mount and links make Stealth's TWB480-FDM.

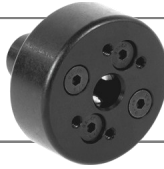

TD100

The Tone Deflector is design to absorb quick impact and aver slanted angle loading.

i2i HEADREST PACKAGES*



i2i280

Niño Extra Small

i2i282

Standard w/Gel



i2i380

Niño Extra Small

i2i382

Standard w/Gel



i2i480

Small i2i

i2i482

Standard w/Gel



i2i481

Sm. w/short arms

i2i483

Standard w/Gel



i2i490

Low Profile

i2i492

Low Profile w/Gel



i2i491

Low Profile

i2i493

Low Profile w/Gel



i2i580

Medium i2i

i2i582

Standard w/Gel



i2i581

Med. w/short arms

i2i583

Standard w/Gel



i2i590

Low Profile

i2i592

Low Profile w/Gel



i2i591

Low Profile

i2i593

Low Profile w/Gel




i2i680

Large i2i

i2i682

Standard w/Gel

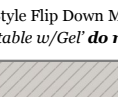


i2i681

Lg. w/short arms

i2i683

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Standard w/Gel



i2i690

Low Profile

i2i692

Low Profile w/Gel



i2i691

Low Profile

i2i693

Low Profile w/Gel

i2i780

Extra Large i2i

i2i782

Standard w/Gel

i2i781

X-Lg. w/short arms

i2i783

Standard w/Gel

i2i790

Low Profile

i2i792

Low Profile w/Gel

i2i791

Low Profile

i2i793

Low Profile w/Gel

i2i740

Adjustable Arms**

i2i742

Adjustable w/Gel

*Headrest Packages come with Headrest Pad, Link Style Flip Down Mounting hardware and Chin Prompt.

**Packages with 'Adjustable Arms' and 'Adjustable w/Gel' do not include a Chin Prompt.

SWING-AWAY OPTIONS

More information available at www.stealthproducts.com



SUS9-B-HW

Stealth Product's Swing-Away hardware offers a sleek, unobtrusive design while featuring an easy, one-hand release mechanism for swinging pads and switches back a full 100°.



SUS9DL-DB

The dual swing away design allows for more attachment sites for facial pads or switches for those clients who need more real estate or extra support.



SUNS9-B-HW

Convenient sizing, versatility, comfort and intuitive function with a clean, elegant design makes Stealth's SUNS9 the best choice when in need for pediatric facial pads and switches.



900

Facial Pad
(4.5" x 2")



910

Facial Spot Pad
(3" x 2")



920

Long Facial pad
(6" x 2")



930

Long Niño Facial Pad
(6" x 1.5")



F8675

Stealth's figure 8 bracket adds real estate for an extra attachment.



JOYSTICK MOUNTS

SM600: Mini Proportional mount
SM610: 1812 mount
SM620: Tash Mini mount
SM630: ASL MEC mount



SP625

Sip & Puff mount



CN600

Swing-away flex Micro-Lite switch.

Go to www.stealthproducts.com to see more of the huge line of Swing-Away products that Stealth offers.